


# ANKLE HUGGERS® AND TOE STRAP INSTALLATION AND USER MANUAL

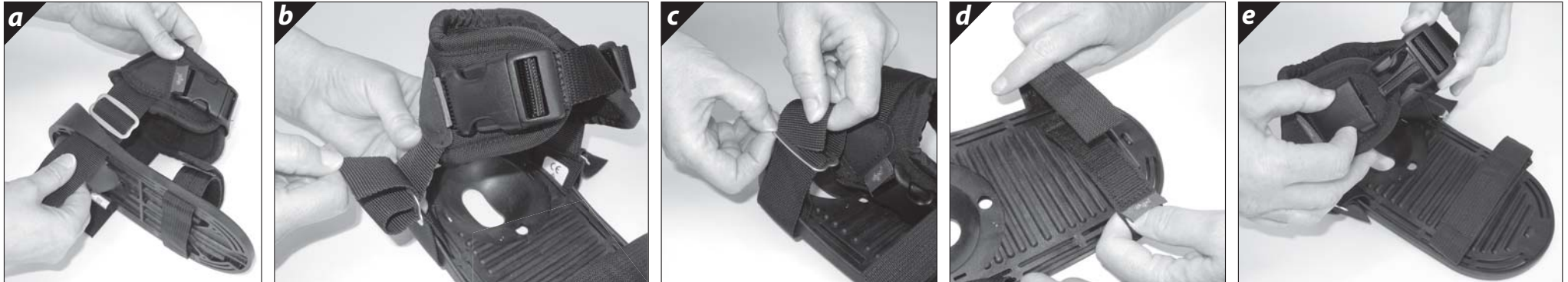

## Package Contents

- Ankle Huggers®** (2) Steel slider  
(2) Toe Straps
- Toe straps** (2) Toe Straps

## Installation

### Ankle Huggers®

- Slide steel slides onto Ankle Hugger® straps.
- Attach the Ankle Hugger® straps to slots at the rear of the footplate, so that the Ankle Hugger® opening faces forward (figure a).
- Adjust the Ankle Hugger® height with the steel slides so that it fits over the ankle (figure b).
- Thread the webbing back over the slide to secure the straps (figure c).

### Toe Straps

- Thread the toe strap through the slots at the front end of the footplate, with the soft loop side of Velcro™ facing up (figure d).

### Ankle Huggers® and Toe Straps

- Placement and tension of straps allows varying degrees of movement.

## Use

- Fasten/unfasten using quick release buckle and Velcro™ as required (figure e).

## Cleaning

- Ankle Huggers® and toe strap: machine wash 40°C with mild detergent. No use of bleach. Line dry. Machine wash, warm, 40° (100°F). Do not bleach. Tumble dry, low temperature, or drip dry. Do not iron. (Placing the product inside a cloth bag during washing helps prevent scratches to the product and the machine).



## ⚠ Safety

- *Chunc wheelchairs are Class 1 medical devices and must not be used to seat any person without prior consultation of a qualified health care professional.*
- *A qualified health care professional should be present when fitting or adjusting the Chunc wheelchair and accessories to an occupant.*
- *The occupant's seating posture and the positioning of adjustable parts and accessories should be monitored closely and adjusted as directed by a health care professional.*
- *Check periodically for signs of wear in the stitching, webbing and pads, if significant wear or damage is found contact Chunc Wheelchairs for replacement or qualified repair.*



Wheelchairs  
for young people

chunc.com

Chunc Wheelchairs  
Specmat Technologies Inc.  
215 Dunavant Drive  
Rockford  
Tennessee, TN 37853, USA

t. (865) 609 1411  
f. (865) 609 1911  
e. sales@chunc.com

Chunc Wheelchairs  
Specmat Limited  
Tarsmill Court  
Rotherwas  
Hereford, HR2 6JZ, UK

t. +44 (0) 1342 377 512  
f. +44 (0) 1432 377 516  
e. sales@chunc.co.uk

The Chunc range of wheelchairs and accessories are designed and manufactured to meet the requirements of the Medical Devices Directive 93/42/EEC for Class 1 medical devices. Specmat Limited holds the British Standard Quality Approval to BS EN ISO 9001:2000 for the design and manufacture of medical devices. All products FDA approved: site registration number 3004709721. Patents pending include EP 03251925, Design Registration includes USD 475952, EC 000080627. Granted Patent U.S. 6,994,364. Chunc is a trademark of Specmat Ltd, part of the HR Smith Group of Companies. © Copyright Specmat Ltd 2007. All rights reserved. All technical information is subject to alteration without notice. Ankle Huggers® is a registered trademark of Bodypoint. Velcro™ is a registered trademark of the Velcro Group of Companies.

THIS USER MANUAL IS AVAILABLE IN LARGE PRINT

For product warranty, please see Chunc user manual: [chunc.com](http://chunc.com)

ANKLE HUGGERS® AND TOE STRAP INSTALLATION AND USER MANUAL issue 1, 12 July 2007, Specmat document no. HRS 1077-04-04, C/N no. 5423

BS EN ISO 9001:2000

